

Allurion

PROGRAM



Start your Allurion
journey today.



*In rare cases, the Allurion Balloon may require endoscopic or surgical intervention for removal.

The program

How it works

Placement

Support

Safety

Results

About Allurion

Next steps

What could losing weight mean to you?

I could...

- ▶ Feel more confident when I look in the mirror...
- ▶ Enjoy my free time more...
- ▶ Improve my food and nutritional choices...
- ▶ Feel free and happier...
- ▶ Improve my physical health...



The gastric balloon program for weight loss and healthier lifestyle habits




Simple and safe

No surgery,
no endoscopy*,
no anesthesia
and placed
in less than 15
minutes

Passes naturally after
approximately
4 months.

A second balloon is placed
after a minimum of a
2-month gap, which also
passes naturally after 4
months.

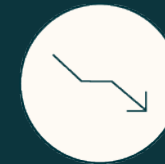

Lifestyle modification

Shift towards
healthier and
sustainable
lifestyle habits

*In rare cases, the Allurion Balloon may require endoscopic or surgical intervention for removal.

The Allurion Program is a new generation weight-loss solution*




*Weight loss treatments are increasingly multimodality, and the Allurion Program can be used in combination with and before / after some other treatments

** Results from common weight loss drugs, are after around 16 months

1. European Association for the Study of Obesity: Three-quarters of adults with obesity have attempted to lose weight in the past year, but most have been unsuccessful. May 2022

How does it work?

- ▶ The Allurion Balloon takes up space in your stomach, leaving less room for food. This is designed to help you feel fuller faster and for longer. It reduces your appetite and portion sizes, aiding weight loss.

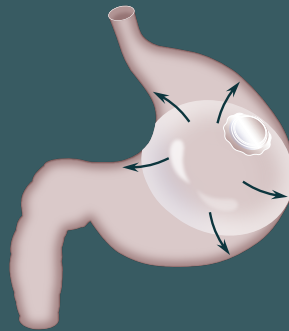

- ▶ The Allurion Smart Capsule is easily swallowed during a 15-minute walk-in appointment
- ▶ The balloon and your lifestyle change program work together to help you create healthier habits that last a lifetime



The only gastric balloon requiring no endoscopy for placement or removal*

What happens on placement day



Scan here to see a video of the Allurion Balloon placement



01 – Swallow

You swallow a Smart Capsule that contains the balloon attached to a thin tube. You will hold the tube in place, or dressing tape can be applied to the tube and affixed to the face to hold it in place. You will have an X-ray to make sure the capsule is in the right position.



02 – Fill


Once in your stomach, your doctor will use the tube to fill the balloon with water.

You will have a second X-ray to confirm the balloon is filled. The doctor will then gently remove the tube and you'll be on your way. With the balloon in your stomach, you can eat less without feeling hungry.



03 – Disappear

At approximately 4 months after placement, the Allurion Balloon self-empties and passes naturally.*

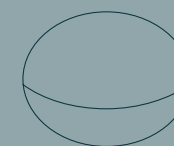

These steps are repeated with each of the two consecutive balloons.

*In rare cases, the Allurion Balloon may require endoscopic or surgical intervention for removal.

Adjusting to life with a balloon

You may experience some discomfort in the days after your placement as your body adjusts to the balloon.

For the first few days after placement you may have nausea, fatigue, abdominal cramping and vomiting. This is completely normal and your nutritionist and doctor can help you manage any symptoms.

You will need to re-introduce food carefully and your nutritionist will support you every step of the way. It is best to not start physical activity (e.g. sports activities) for at least one week after placement, when any symptoms you may have had are gone.



No special diets, fad foods or extreme exercise regimes.
Just personalized, achievable and enjoyable goals that are right for you

Your balloon helps you

- ▶ **Feel full faster** - eat smaller portions
- ▶ **Feel full for longer** - reduce your appetite

Support from your nutritionist helps you

- ▶ Kick-start healthier lifestyle habits, shift your mindset and empower you to maintain your weight loss over the long term
- ▶ Feel supported throughout your journey

A fully supported program for sustainable long-term weight loss



Smart, simple, and innovative: A balloon redefined by design

01

Unlike endoscopic balloons, the Allurion Balloon conforms to the shape of the stomach – an oval shape like a grapefruit, with two flat surfaces

02

Made from polyurethane, a material that is much thinner and more flexible than the silicone material in endoscopic balloons



03

Unlike endoscopic balloons, the Allurion Balloon, being malleable, flexes and adapts to the shape of your stomach

04

The release valve is made from thin film with a hole sealed by a degradable filament that weakens in the balloon's internal environment, breaking over time to empty the balloon at a set point

In the AUDACITY Study, the Allurion Balloon demonstrated a proven safety profile and was well-tolerated.*

96% of all device or procedure-related adverse events were mild or moderate, with the most common being nausea.¹

MOST COMMON ADVERSE EVENTS

Nausea

Abdominal Pain

Vomiting

Abdominal Discomfort

Diarrhea

Heartburn

Only 3.0% of subjects had a device or procedure-related serious adverse event.¹


¹ The AUDACITY Study. Data on File (2025).

* In rare cases, the Allurion Balloon may require endoscopic or surgical intervention for removal.

The Allurion Balloon has the lowest rate of serious adverse events compared to FDA approved liquid-filled gastric balloons.*

Allurion subjects were less likely to experience a device or procedure related serious adverse event than subjects treated with another liquid-filled gastric balloon.¹

Serious Adverse Event Rates for Liquid-Filled Intra-gastric Balloons

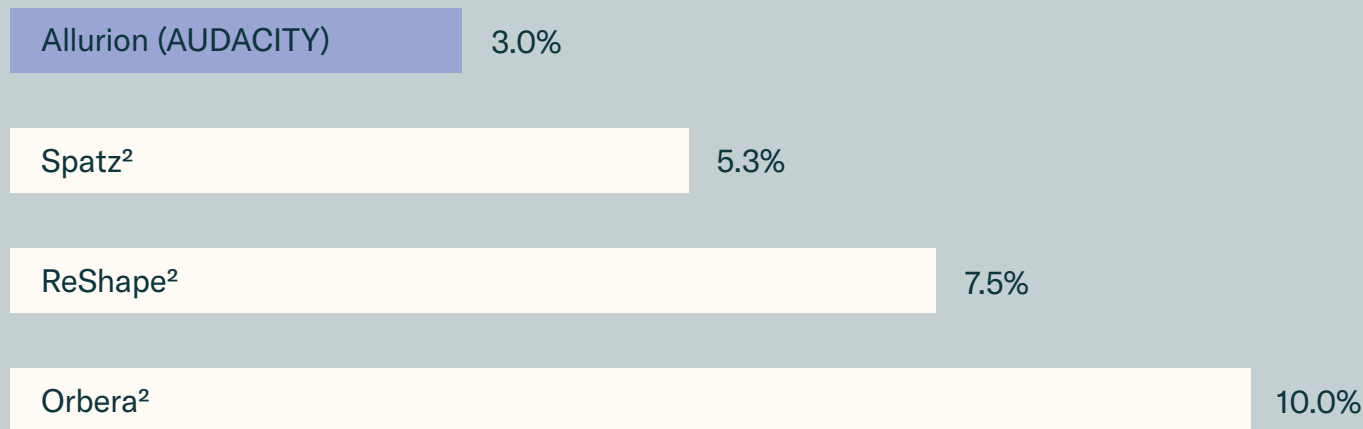

¹ The AUDACITY Study. Data on File (2025).

² Data from SSEDs of the devices.

* In rare cases, the Allurion Balloon may require endoscopic or surgical intervention for removal.

Effective weight loss requires comprehensive care. That's why the Allurion Program offers much more than just a balloon.



10+ month
Allurion Program

Included in the **program**



2 balloons



Medical support



Nutrition coaching
10+ months

A Healthier, Happier You*

Allurion subjects reported significant improvements in:

- Overall quality of life
- Physical function
- Self-esteem

Higher scores on the IWQOL-L questionnaire mean a better quality of life. After 48 weeks, people in the study who were in the AGBS Group showed bigger improvements in their overall quality of life and in specific areas like physical function and self-esteem, compared to those in the Control Group. These differences were not just by chance—they were statistically significant.

*The AUDACITY Study. Data on File (2025).

How Eating Patterns Shift During the Program*

Both groups of patients in the study also answered questions regarding their eating behaviors at baseline (before treatment) and at Week 32. In the Allurion Balloon group, there were significant improvements in cognitive restraint and decreases in emotional and uncontrolled eating.



*The AUDACITY Study, Data on File (2025).


Allurion is an International Success Story



Founded in 2009 with
10+ years of R&D



200,000+ balloons
distributed in over
78 countries
(October 2025)

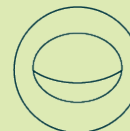

Extensive body of
scientific evidence



Designed and
manufactured in the USA



Allurion

PROGRAM

Take the next
step in your
weight-loss
journey

Start your Allurion Program today

